THUASNE

ImmoClassic

- fr** Écharpe évolutive d'immobilisation d'épaule
- en** Adjustable shoulder immobilisation sling
- de** Erweiterbares Tuch zur Ruhigstellung der Schulter
- nl** Sling voor immobilisatie en progressieve opbouw van de schouder
- it** Sciarpa evolutiva di immobilizzazione della spalla
- es** Cabestrillo evolutivo para inmovilización del hombro
- pt** Suporte evolutivo de imobilização do ombro
- da** Halstørklæde foranderlig skulderimmobilisering
- fi** Olkapään modulaarinen immobilisaatiokantoside
- sv** Band för att immobilisera axeln
- el** Προσαρμόσιμος φάκελος ακινητοποίησης ώμου
- cs** Modulární imobilizační ramenní závěs
- pl** Szal do unieruchomienia barku z możliwością adaptacji
- lv** Regulējamā pleca imobilizācijas soma
- lt** Pritaikomas peties imobilizavimo įtvaras
- et** Öllaliigest fikseeriv reguleeritav side
- sl** Nastavljiva zanka za imobilizacijo rame
- sk** Nastaviteľný záves na imobilizáciu ramena
- hu** Moduláris vállrögzítő heveder
- bg** Бандаж за обездвижване на рамото, адаптируем
- ro** Orteză ajustabilă „tip eşarfă” de imobilizare a umărului
- ru** Регулируемая повязка для обездвиживания плеча
- hr** Šal za imobilizaciju ramena, produljivi
- zh** 有适应能力的肩部固定围巾
- ar** وشاح لتثبيت الكتف قابل للتطوير

ImmoClassic *

- fr** Avec coussin d'abduction
- en** With ergonomic abduction pillow
- de** Mit Abduktionskissen
- nl** Met abductiekussen
- it** Con cuscino di abduzione
- es** Con almohadilla de abducción
- pt** Com almofada de abdução
- da** Med abduktionspude
- fi** Loitontajatyynyllä
- sv** Med abduktionskudde
- el** Με μαξιλάρι αναγωγής
- cs** S abdukčním polštářem
- pl** Z poduszką odwodzącą
- lv** Ar abdukcijas polsteri
- lt** Su atskiriamąja pagalvele
- et** Koos abduktioonipadjaga
- sl** Z blazino za abdukcijo
- sk** S abdukčným vankúšom
- hu** Abduktációs párnával
- bg** С възглавничка за абдукция
- ro** Cu pernă de abducție
- ru** С отводящей подушкой
- hr** S abdukciskim jastukom
- zh** 带有外展垫子
- ar** مزود بوسادة إبعاد

1	< 35 cm
2	> 35 cm

© Thuasne - 6170601 (2022-08)
© Studio Caterin

THUASNE
120, rue Marius Aujan
92300 Levallois-Perret
France

www.thuasne.com

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ADJUSTABLE SHOULDER IMMOBILISATION SLING

Description/Destination

This device is a shoulder sling with an abduction pillow for immobilizing the shoulder with the hand beneath the chest or in a neutral position.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyester - polyurethane - polyamide - cotton - POM (buckle).

Properties/Mode of action

Adjustable strap system for immobilizing the shoulder with the hand beneath the chest or in a neutral position.

Adjustable foam pad for comfort and protection of the neck.

ImmoClassic +:

Neutral position with a 20° ergonomic abduction pillow for specific immobilisations.

Indications

• Shoulder and/or elbow injuries or inflammatory conditions.

• Conservative treatment for non-surgical proximal humeral fractures.

• Shoulder dislocation.

• Non surgical fractures of the clavícula and/or scapula.

• Post-operative immobilisation (such as after rotator cuff surgery or dislocation reduction...).

• Shoulder subluxation or dislocation prevention (joint instability).

• Support for paralytic upper limb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in certain post-operative conditions (multiple fractures, pin insertion, open fractures, etc.).

Do not use in the event of complex or surgical shoulder fractures, humeral fractures or rib cage fractures.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product when driving a vehicle.

Prolonged immobilisation can sometimes cause muscle weakness. If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve immobilisation without restricting blood circulation.

Wear the device above thin clothing.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

The help of another person is required to put on this product.

Bilateral product: the sling can be adjusted for right-hand or left-hand use by reversing the removable straps.

You can take off the removable self-fastening tab ☺ and cut the straps ☹ and ☺ to better adjust the product to your body shape.

Make sure you cut the straps straight across and reposition the self-fastening tabs correctly. Once these are in position, test whether the end of the strap remains in place by pulling on it.

ImmoClassic:

❶ Place the forearm in the sleeve ☹ so that the elbow is correctly positioned and then slip the removable cut-to-size strap ☹ around the neck.

❷ Adjust the height of the sleeve using the strap ☹ if necessary and then pass the removable strap ☺ around the waist and fix it to the area provided for the purpose. Position it using the self-fastening tab ☺ on the sleeve ☹ to immobilize the forearm against the body. The recommended angle for the elbow is 90°.

❸ Tighten the sling around the forearm using the self-fastening tab ☹.

ImmoClassic +:

❶ Place your forearm in the sleeve ☹ so that the elbow is correctly positioned, then slip the strap ☹ around your neck.

❷ Place the protruding part of the pillow ☹ on the front of the body and the narrowest part against the body by the elbow. Attach the sleeve ☹ to the pillow.

❸ Adjust the height of the sleeve using the strap ☹.

❹ Place the strap ☺ around the waist. Use the self-fastening loops ☺ to fix the strap ☹ to the pillow. Tighten the sling around the forearm using the small self-fastening strap ☹.

To take off the product, remove the small self-fastening strap ☹, detach the strap ☺ followed by the strap ☹ and remove the arm from the sleeve ☹.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Remove the self-fastening tabs and the straps before washing. Sleeve and abduction pillow machine-washable at 30°C (delicate wash). The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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