
THUASNE

V1213714

fr	Barre d'appui rabattable	2
en	Folding grab bar	3
de	Klappbarer Duschgriff	4
nl	Opklapbare douchebeugel	5
it	Sostegno d'appoggio doppio ribaltabile	6
es	Barra de apoyo abatible	7
cs	Sklopná opěrná tyč	8
pl	Składany uchwyt	9
sk	Sklopné oblúčkové držadlo	10
hu	Felhajtható kapaszkodó	11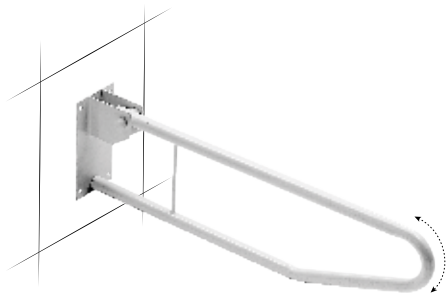

BARRE D'APPUI RABATTABLE

Généralités

Dimensions : 72,5 x 26,5 x 22 cm

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General information

Dimensions: 72.5 x 26.5 x 22 cm

These are usually fitted on each side of the toilet, at a height of around 80 cm.

Maximum user weight: 120 kg

People who are able to walk use the bars as support when sitting down or standing up. For people going to the bathroom in a transit chair, one of the bars can be raised to allow easy access to the toilet while the other is used as support when transferring the user.

Parts supplied

Polyamide screws and washers. Use fixings suitable for the wall type. If in doubt, it is preferable to have the bar fitted by a professional.

Fitting instructions

- Mark the position of the holes on the wall.
- Drill the holes in the wall and insert the plugs.
- Screw the bar into the wall using the screws supplied in the fixing kit.

Warranty

Product guaranteed one year against any abnormal deterioration in routine normal conditions of use, on condition that the product is used according to the recommendations in the instructions.

The warranty is not valid in the event of use of products containing chlorinated or non-chlorinated solvents. It only covers replacement of the product, on condition that evidence of the delivery date is provided.

Do not use a damaged or worn product. Check the stability of fixings carefully before use.

Clean the product regularly using a non-abrasive detergent.

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©Thuasne - 2141602 (2021-06)