THUASNE

Ligaflex Boxer®

fr	Attelle d'immobilisation poignet-main-doigt.....	5
en	Wrist-hand-finger immobilisation splint.....	6
de	Immobilisierungsschiene für Handgelenk/Hand/Finger.....	7
nl	Immobilisatiebrace voor pols, hand en vinger.....	8
it	Tutore di immobilizzazione polso-mano-dita.....	9
es	Férula de inmovilización muñeca-mano-dedo.....	10
pt	Tala de imobilização para o punho/mão/dedo.....	11
da	Skinne til immobilisering af håndled-hånd-finger.....	12
fi	Ranteen, kämmenen ja sormien immobilisaatiolasta.....	13
sv	Immobiliseringsskena till handled-hand-finger.....	14
el	Νάρθηκας ακινητοποίησης καρπού-χειριού-δακτύλου.....	15
cs	Imobilizační ortéza zápěstí, ruky a prstů.....	16
pl	Unieruchamiająca orteza nadgarstka, ręki i palca.....	17
lv	Plaukstas locītavas, plaukstas un pirkstu imobilizācijas šina.....	18
lt	Riešo, plaštakos ir pirštų imobilizavimo įtvaras.....	19
et	Randme-käe-sõrme fikseerimise lahas.....	20
sl	Opornica za imobilizacijo zapestja, roke in prsta.....	21
sk	Ortéza na znehybnenie zápästia, ruky a prstov.....	22
hu	Csukló-kéz-ujjrögzítő sín.....	23
bg	Шина за обездвижване китка - ръка - пръст.....	24
ro	Atelă de imobilizare încheietura mâinii-mână-deget.....	25
ru	Шина для иммобилизации запястья-руки-пальца.....	26
hr	Udlaga za imobilizaciju ručnog zgloba-šake.....	27
zh	腕-掌-指部固定支具.....	28
ar	جيرة لتثبيت الرسغ واليد والأصابع.....	31

	1	13 - 15 cm
	2	15,5 - 17,5 cm
	3	18 - 20 cm
	4	20,5 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonferenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
sv	Ranteen ympärysmitta
fi	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstas locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
zh	Obujam ručnog zgloba
hr	腕圍
ar	محيط الرسغ

Immobilisation
Immobilisation
Ruhigstellung
Immobilisatie
Immobilizzazione
Immobilización
Imobilização
Støtte
Immobilisoi
Immobilisering
Ακντοποίηση
Znehybnění
Unieruchomienie
Imobilizacija
Imobilizavimas
Liikumatuks muutmine
Imobilizacija
Imobilizácia
Rögzítés
Обездвиждане
Imobilizare
Иммобилизация
Imobilizacija
固定
التثبيت

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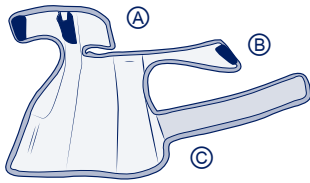
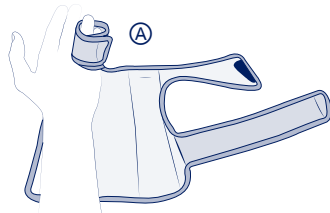
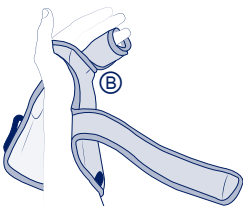
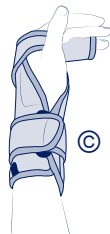
WRIST-HAND-FINGER IMMOBILISATION SPLINT

Description/Destination

This device is a wrist-hand-finger splint providing immobilisation of the 4th and 5th fingers.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - polyurethane - elasthan - polyester - aluminium.

Properties/Mode of action

Immobilisation of the wrist and 4th and 5th fingers via 2 rigid and conformable stays.

Palmar stay pre-formed at 20°; dorsal stay pre-formed at 90° at the fingers.

The brace allows full opening for easier positioning of the hand and the other fingers.

Separation between fingers to protect against any possible friction on the immobilised fingers.

Indications

Post-operative/post-traumatic immobilisation of the 4th and 5th metacarpals.

Non-surgical 4th and 5th metacarpal fracture.

Residual symptoms post fractured or sprained finger (of 4th and 5th).

Following strict plaster cast immobilisation.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in case of unstable fractures.

Precautions

Verify the product's integrity before every use. Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Place the brace on a flat surface ①, position the hand with the palm facing upwards, gently insert the 4th finger into the loop (finger stall) provided. ②

Fold down the strap ③ to immobilise the 4th and 5th fingers together, which will lock the top part of the brace, at the fingertips. ④

Turn the hand onto its side and fasten the intermediate strap ⑤. Ensure that the palmar stay is positioned correctly against the palm. ⑥

Fasten the final strap ⑦, inserting it into the buckle provided, and tighten until sufficient support is obtained. ⑧

To prevent any constrictive problems, take care not to overtighten the straps. Do not hesitate to readjust the brace if required.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30°C (delicate programme). If possible use a washing net. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Polyamid - Polyurethan - Elasthan - Polyester - Aluminium.

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©Thuasne - 2027101 (2021-05)