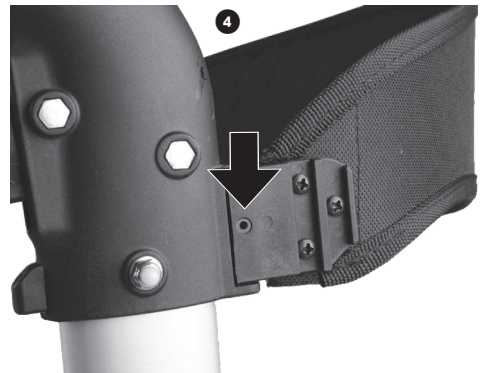
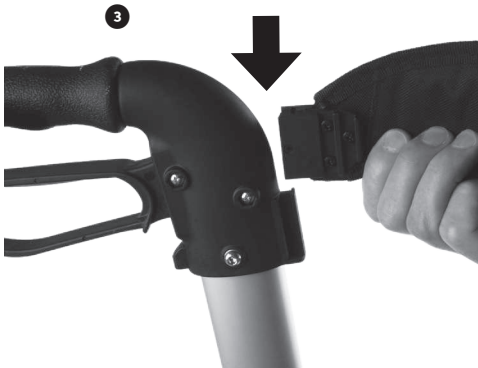
**THUASNE**

# New Design

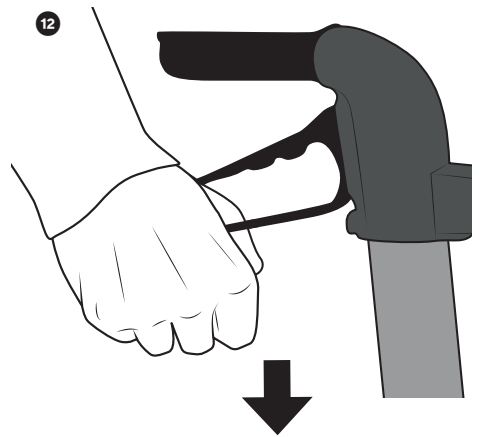
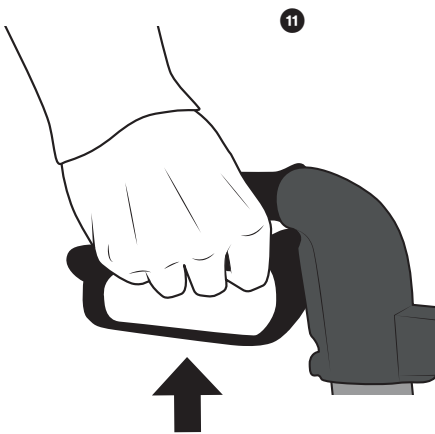
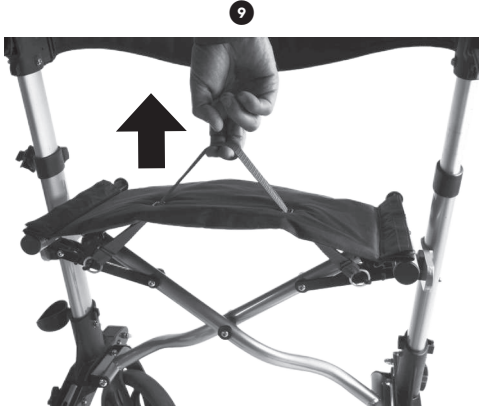
W2230

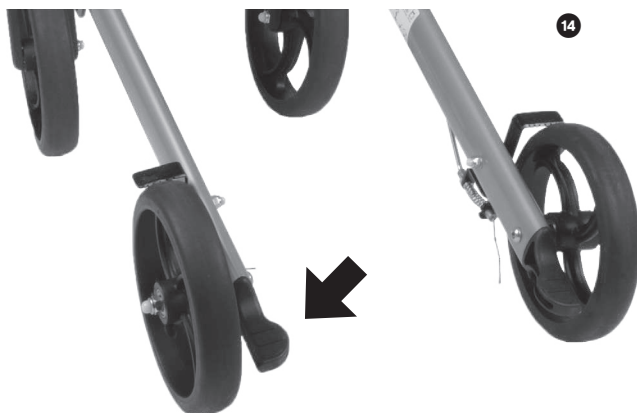
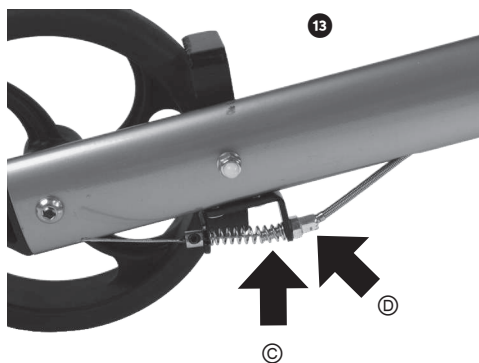
fr	Rollateur	6
en	Rollator	7
de	Rollator	9
nl	Rollator	11
it	Rollator	13
es	Andador	14
cs	Chodítka	16
pl	Podpórka	18
sk	Štvorkolka	19

**MD**

<b>fr</b>	Dispositif médical
<b>en</b>	Medical device
<b>de</b>	Medizinprodukt
<b>nl</b>	Medisch hulpmiddel
<b>it</b>	Dispositivo medico
<b>es</b>	Producto sanitario
<b>cs</b>	Zdravotnický prostředek
<b>pl</b>	Wyrób medyczny
<b>sk</b>	Zdravotnícka pomôcka

<b>fr</b>	Lire attentivement la notice
<b>en</b>	Read the instruction leaflet carefully
<b>de</b>	Lesen Sie die Gebrauchsanweisung sorgfältig durch
<b>nl</b>	Lees voor gebruik de gebruiksaanwijzing aandachtig door
<b>it</b>	Leggere attentamente le istruzioni
<b>es</b>	Leer atentamente las instrucciones
<b>cs</b>	Pozorně si přečtete návod
<b>pl</b>	Należy dokładnie przeczytać instrukcję
<b>sk</b>	Tento návod si pozorne prečítajte

# ROLLATEUR NEW DESIGN

## Description/Destination

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### Description/Destination

4-wheel rollator.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It also has a seat with a backrest for taking short rests. It can be used inside and outside.

### Main components

Frame: aluminium

Handles: PU

Seat (cover): nylon

Backrest (cover): nylon

Wheels/tires: thermoplastic elastomer

Shopping bag: nylon/mesh

### Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.

Degenerative conditions of the lower limbs.

### Contraindications

Not suitable for patients weighing > 130 kg.

Do not use in the event of known allergy to any of the components.

Do not use in the event of severe perceptual function deficit or severe balance disorders.

## Precautions

### Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Before using this product, check that:

- it is stable and rigid,
- the brakes are working properly,
- the wheels are in good condition and turn freely,
- no components are incorrectly attached or not properly screwed in place.

The health care professional who supplied the product must make sure that it is appropriate for the patient.

It is recommended that a health care professional assembles and adjusts the device and supervises the first use. In particular, it is essential that they check and adjust the tension of the brake cables.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Do not use the device for any use other than its intended use: risk of injury or damage to equipment.

Never allow children to play with the product.

Beware of the risks of pinching when handling the device.

Always completely unfold the rollator before use.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven, sloping or unstable surfaces (gravel, sand, leaves, etc.).

Do not use on stairs or escalators.

Do not leave exposed to direct sunlight for a long period of time. Check the temperature of the device surfaces, particularly before sitting down on the seat: risk of burns.

### Undesirable side-effect

- Risk of discomfort in the event of incorrect adjustment of the device.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Instructions for use

### Unfolding

#### 1. Installing the wheels

Place the rollator on one side. Unscrew and remove the rear wheel support knob nut, then unfold the rear wheel support until the holes are aligned. Put the knob nut back in place, making sure it is fully tightened.

Proceed in the same manner for the front wheel. Then place the rollator on the other side and repeat the same operations.

#### 2. Opening the rollator

Place the rollator on its wheels, and pull the handles apart. Push on the side tubes of the seat until the mechanism engages, to lock the rollator in the open position. ❶

#### 3. Installing the backrest

Position the backrest correctly ❷, then slide the ends of the backrest downwards into the rollator supports provided for that purpose ❸.

Then fix the backrest in place by fully tightening the screws. ❹

#### 4. Fitting the shopping bag

The shopping bag should be installed at the front of the two supports on either side of the seat. ❺

#### 5. Fitting the stick holder

Clip part ❶ to the bottom of the left or right tube. Part ❷ should be inserted into the notch at the top of the tube, just below the handle. ❸

### Adjusting the handle height

The height of the device must be adjusted with reference to the height of the patient. Incorrect adjustment can impair performance and place too much strain on the user. The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes. ❷

- Press the push-buttons while pulling the handle in order to adjust the height. Make sure that the handles are fixed at the same height on both sides. Check that the push-buttons are properly engaged before use. ❸

### Folding

- To fold up the rollator, pull the seat strap upwards. ❹  
Once folded, the rollator stands on its own.
- To fold up the rollator completely for easier transport, place the handles in the low position then fold up the rear wheel supports. Fold up the front wheels if necessary. ❺

### Braking system

- Quick braking  
To brake, hold on to the handles and squeeze the brake levers. ❶
- Parking brakes  
To immobilize the rollator (e.g., before sitting down), carefully push the brake levers down until they lock (click). ❷  
To start moving again, gently squeeze the levers to release the brakes.
- Adjusting the brakes  
- If braking is not sufficient, loosen the locknut ❸ and turn the adjustment screw ❹ anticlockwise. ❸  
- If braking is too hard, proceed as above but turn the screw clockwise. ❸ Remember to tighten the locknut again after adjusting to the optimal setting.

### Curb-climber:

- This rollator is fitted with a curb-climber to make it easier to negotiate obstacles. ❶

### Important warnings

Before using the seat:

- check that the rollator is on a flat, stable surface,
- it is essential for the brakes to be in the "parking brakes" position.

Do not sit at an angle or lean to the side or back.

The rollator seat is only intended for taking short rests.

Do not attach handbags, carrier bags or any objects other than those supplied with the device: risk of tipping.

The device must not be used as an aid to get up from the sitting position: risk of unbalancing.

## Maintenance and care

### Maintenance

If any part of the device becomes detached, loose or visibly damaged, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

### Care

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handles or the wheels with greasy substances: risk of slipping.

If necessary, use a mild contact disinfectant or 70° alcohol. In an institution, follow the recommendations of the hygiene manager.

## Storage

Store at room temperature, in a dry, ventilated place.

## Disposal

Dispose of in accordance with local regulations.

## Technical characteristics

Maximum user weight  130 kg max		
Handle adjustment height  79 - 91,5 cm Steps of 2,5 cm	Total width  64,5 cm	Total depth  70 cm
Folded rollator width  23 cm	Total weight with shopping bag  7,9 kg	Seat height  54 cm
Seat width  45 cm	Seat depth  22,5 cm	Shopping bag load  5 kg max

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- Si le freinage n'est pas suffisant, desserrer le contre-écrou © et tourner la vis de réglage Ⓧ dans le sens antihoraire. 13